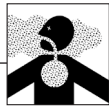
Warnung!

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BSI Group The Netherlands B.V.

Say Building, John M. Keynesplein 9, 1066 EP, Amsterdam, Netherlands

Notified body: 2797

Importeur:

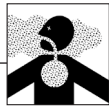
Franz Mensch GmbH

Werner-von-Siemens-Str. 2

86807 Buchloe Germany, www.franz-mensch.de

Warning!

Abuse can lead to disease and death

For proper use, please read the instructions carefully. Prior to use, check manufacturer's expiry date, discard if expired.

This mask is suitable for use against solid, water and liquid based aerosols according to EN 149:2001+A1:2009. Masks are classified according to three categories (FFP1, FFP2 or FFP3).

Selection of respiratory mask

Filter category FFP= Filtering Face Piece	MAK	Use	Comments
FFP1 / P1	up to MAK 4	Protection against coarse and dry particles which are dangerous for one's health and aerosole; e.g.: pollen, coarse allergens ...	Masks with exhalation valve: • Easier exhalation • Reduction of temperature with in the mask, therefore no breathing
FFP2 / P2	up to MAK 10	Protection against dry particles which are dangerous for one's health and aerosole; e.g.: dust from a building site or from polishing	Masks according to EN 149:2001 – they have a softer nose pad made of a moisture repellent material, therefore • higher comfort • no breathing film on the (safety) glasses
FFP3 / P3	up to MAK 30	Protection against very small and dry particles which are dangerous for one's health and aerosole; e.g.: chemicals industry, bacteria, viruses...	

Protection standards comply with MAK. This standard corresponds to the concentration on the workplace an employee may be exposed to, without suffering from any health risks.
FFP1 = small particle quantity FFP2 = medium particle quantity – filters 94% of the particles
FFP3 = important particle quantity – filters 99% of the particles

NR = non reusable D = passed Dolomite clogging test

Instruction for use:

- Before using the respiratory protection mask, ensure that the mask meets the requirements of the environment. Follow application instructions and recommendations from safety officers. Failure to follow instructions may endanger your health or life. Never hesitate to ask a professional (a person responsible for occupational safety, e.g., an occupational physician) to help you determine the protection you need.
- Observe national regulations and laws regarding the use of personal protective equipment.
- This mask is a disposable mask. It must be discarded after a maximum of 8 hours of continuous use. There is no way to maintain its protective effect after use. At the first signs of deterioration or if breathing resistance becomes too high, the product must be replaced.
- Use only in sufficiently ventilated rooms with an oxygen content >17%; the mask does not supply oxygen.
- In case of dizziness, breathing difficulties, or nausea, leave the workplace immediately and go to a well-ventilated area to avoid more serious conditions.
- Never modify this mask, which has been developed and approved for your safety. Do not attempt repairs.
- Incorrect use of the mask or failure to observe the warnings reduces its protective effect and may lead to illness or permanent health damage.

Not suitable for:

- Don't use this mask if you don't know with precision the nature of the toxic that you can encounter. Don't use it notably if you are confronted to explosive products. This mask must not be used in explosive atmospheres.
- The mask does not protect against gases or vapors, or solvents.
- Non-vented rooms, containers, pits, ducts, etc. must not be accessed with half masks.
- The mask is less suitable for people with beards or individuals with particularly pronounced facial features, as this may affect the seal. People with different face structures should take extra care to place the mask in such a way that there are no airways.

Prior to use:

Use the mask only if it is not damaged. Verify that the nose slip is not piercing through the filter medium. Open the mask and make sure that the mask has 2 elastics. Take a mask in one hand (the external side of the mask in the inside of your hand). Form the nasal barrette using the inch and the index.

With the other hand, take one of the two elastic bands, stretch it carefully and place it over your head securing it under the ears. Take the second elastic band pass it over your head by the summit of the head. Adjust the mask to your face by pressing the nosepiece against the bridge of your nose. Shape the nosepiece with your thumb and index finger until it fits your nose precisely and no air can enter between the mask and your face.

Then take the time to adjust the entire mask to each contour of your face to ensure that no unfiltered air can pass through at any point. The actual effectiveness of the mask—and thus the user's health—depends on this adjustment.

To check the seal, cup the mask with both hands and exhale forcefully. If air escapes, check the fit of the mask and the elastic straps and adjust if necessary.

This mask has been designed to allow the wearing of glasses: we recommend placing the glasses on the bridge of the nose above the mask.

Storage:

 Temperature range for storage and transportation

 Humidity range for storage and transportation

- Keep your mask at temperatures between -25°C and +40°C and at a maximal humidity level of <70% during transport and storage operations. Storage life is 5 years from the date of manufacture (please refer to the smallest packaging). Then mask has been expired and needs to be discarded.
- In any case and in spite of the whole care brought to this respiratory protective mask by the manufacture, it is the users responsibility (and in no case the manufacturer's responsibility) to ascertain the adequacy of the mask with the environmental features it will be stored and eventually be used in. The manufacturer cannot guarantee a precise use length considering the absence of knowledge of the environment in which the mask will have been stored since its exit of the factory and the features of environment in which it is going to be used (temperature, pressure, hygrometry, concentration in particles to filter, morphology of the user, well stocked efforts...).
- Dispose of in accordance with local regulations.

Marking:

Examples of possible markings on the filtering face mask:

SY 8002 FFP2 NR ← 1
EN 149:2001+A1:2009 ← 3
CE 2797 ← 2

What the symbols mean:

- 1 Example article and model number, FFP category of the mask
NR: Mask is not reusable
D: Passed dolomite dust test
- 2 The CE marking indicates compliance with the requirements of PPE Category III according to European Regulation (EU) 2016/425.
- 3 defines EN 149:2001 + A1:2009 general requirements for classification of filtered half masks for protection against particles.

 Herstellungsdatum: TT/MM/JJJJ

 Zu verwenden bis: TT/MM/JJJJ

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 Gebrauchsanleitung beachten

The full text of the EU declaration of conformity is available at the internet address:
www.shineyamask.com

This product has been approved by the Notified Body

BSI Group The Netherlands B.V.
Say Building, John M. Keynesplein 9, 1066 EP, Amsterdam, Netherlands
Notified body: 2797

Manufacturer:

Zhangjiagang ShineYa Sanitary Products Co., Ltd
No. 198, ShuangLong Road, FengHuang Town, Zhangjiagang
City, JiangSu Province, China, www.shineyamask.com

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